

# An Amniotic Membrane Allograft for Spinal Surgical Barrier Applications

## AmnioFix<sup>®</sup>



### Anterior Spinal Applications

Anterior Cervical Discectomy and Fusion  
Anterior Lumbar Interbody Fusion

### The AmnioFix<sup>®</sup> Amniotic Membrane Barrier's use on soft tissue:

Reduces Inflammation  
Reduces Scar Tissue Formation  
Enhances Healing of Surgical Site

### Posterior Spinal Applications

Laminectomy\*  
Posterior Lateral Fusion\*

\*Including Minimally Invasive

MiMedx<sup>™</sup>  
GROUP

AmnioFix® is a composite amniotic tissue membrane minimally manipulated to protect the collagen matrix and its natural properties.



### Growth Factors present in AmnioFix\*:

- Epidermal Growth Factor (EGF)
- Transforming Growth Factor Beta (TGF-β)
- Fibroblast Growth Factor (FGF)
- Platelet Derived Growth Factors (PDGF) A&B

### Reasons for Using AmnioFix® in Spinal Applications:

- Reduce scar tissue formation near or on dura
- Reduce inflammation in the surgical site
- Enhances healing
- Acts as a barrier to provide a dissection plane

### Tissue Offering

ITEM NUMBER	DESCRIPTION & SIZE
APS-5160	16 mm diameter disc
APS-5230	2 cm x 3 cm graft
AAS-5330	3 cm x 3 cm graft
APS-5440	4 cm x 4 cm graft
AAS-5460	4 cm x 6 cm graft

### Ordering Information

Customer Service: (866) 477-4219

Email: customerservice@mimedx.com

\*Confirmed by HPLC tests of sterilized AmnioFix.

60 Chastain Center Blvd., Suite 60, Kennesaw, GA 30144, USA

AmnioFix® is a trademark of MiMedx Group, Inc.

AmnioFix® is processed by Surgical Biologics, a MiMedx Group Company.

©2011 MiMedx Group, Inc. All rights reserved.

### Dry Application† for Spine Procedures

1. Cut the AmnioFix to the appropriate size.
2. Ensure all excess blood/fluid is removed from the surgical site.
3. Lay the AmnioFix so SB can be read on the desired surgical site.
4. The AmnioFix will become tacky to the site. If surgeon needs to reposition, just add a few drops of sterile saline or water.
5. Suture the AmnioFix to soft tissue if desired.
6. To make the AmnioFix more malleable, add a few drops of saline to hydrate the graft. The AmnioFix may be contoured to the underlying anatomy.
7. Do not suction near the AmnioFix.
8. Complete surgical procedure and close.

†AmnioFix may be applied dry or wet

